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Adhesive remover

1. Indication

- For efficient removal of excessive and residual adhesives without damaging the tooth substance after completion of an orthodontic treatment.
- For the very fine finishing of composite fillings.

2. Contraindication

Due to their special blade geometry the instruments are not suited for cutting tooth substance.

3. Clinical sequence/proper use

The adhesive removers are used without pressure. Always supply plenty of water coolant (at least 50 ml/min.)

Recommended speed for shank 314:

- Use in the red contra-angle at an optimum speed of O_{rot} 120,000 rpm
- Use in the turbine at an optimum speed of O_{rot} 160,000 rpm

Recommended speed for shank 204:

- Use in the micro-motor/contra-angle at approx. O_{rot} 40,000 rpm

Apart from this general information, please observe the individual speed indicated on the packaging. The instruments must be used with utmost care to avoid damage to the dental enamel. After complete removal of the adhesive the enamel will be shiny.

4. Further treatment

If necessary, finish the tooth surface with polishers.

5. Cleaning, disinfection and sterilization

The instruments are to be disinfected with anti-corrosive disinfection and cleaning agents for rotary

instruments. For recommendations for use (immersion time, concentration, suitability) of disinfection and cleaning agents see instructions of the manufacturers of these agents. Disinfecting and cleaning agents must be rinsed off thoroughly with water and the instruments must be dried carefully (preferably with compressed air according to the recommendations of the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute). Do not store instruments for a longer period in wet or humid condition. Make sure that they do not come in contact with each other during ultrasonic cleaning. The instruments can be reprocessed in the thermo disinfectant, provided that the agent used is suitable for rotary instruments (mildly alkaline detergent). Subject cleaned instruments to a visual examination. Damaged or blunt instruments must be rejected and their use discontinued. Sterilization is carried out in the autoclave at 134°C. The recommendations provided by the manufacturer of the respective device must be observed. Discard any corroded instruments. The operator of medical products is responsible for seeing that proper treatment is carried out by qualified personnel, using the appropriate materials and suited equipment, as recommended by the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute. Work instructions with regard to proper reprocessing of instruments according to DIN EN ISO 17664 can be downloaded from our web site www.drendel.com or requested from the manufacturer Drendel+Zweiling DIAMANT GmbH.

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instruments. For recommendations for use (immersion time, concentration, suitability) of disinfection and cleaning agents see instructions of the manufacturers of these agents. Disinfecting and cleaning agents must be rinsed off thoroughly with water and the instruments must be dried carefully (preferably with compressed air according to the recommendations of the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute). Do not store instruments for a longer period in wet or humid condition. Make sure that they do not come in contact with each other during ultrasonic cleaning. The instruments can be reprocessed in the thermo disinfectant, provided that the agent used is suitable for rotary instruments (mildly alkaline detergent). Subject cleaned instruments to a visual examination. Damaged or blunt instruments must be rejected and their use discontinued. Sterilization is carried out in the autoclave at 134°C. The recommendations provided by the manufacturer of the respective device must be observed. Discard any corroded instruments. The operator of medical products is responsible for seeing that proper treatment is carried out by qualified personnel, using the appropriate materials and suited equipment, as recommended by the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute. Work instructions with regard to proper reprocessing of instruments according to DIN EN ISO 17664 can be downloaded from our web site www.drendel.com or requested from the manufacturer Drendel+Zweiling DIAMANT GmbH.

6. Safety and liability

Worn and damaged instruments (bent instruments etc.) have to be rejected out and replaced by new ones. The above mentioned recommendations with respect to handling and cooling are to be strictly observed. The instruments should only be used for the intended application. Non-observance of these safety recommendations may lead to damage of the handpiece or injury. In case of contributory negligence by the user, Drendel+Zweiling DIAMANT GmbH partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

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© 313214 – Rev 2017-06