## Veneer Set Coffret pour facettes

An essential condition for a successful restoration with ceramic veneers is for example a systematic, conservative preparation. This poses a particular challenge for the dentist: On one hand, a certain amount of material needs to be removed, on the other hand, care has to be taken not to penetrate too deeply into the enamel.

In response to this, a new set was developed that contains all the required instruments.

The set comprises newly developed depth markers (868B) to define the reduction depths (0.3 and 0.4 mm), resulting in a final preparation depth of 0.4 or 0.5 mm after finishing. In addition, the set contains tapered medium grit (100  $\mu\text{m}$ ) diamond abrasives (868) and fine grain (30  $\mu\text{m}$ ) diamond finishing instruments (8868) to match the new depth markers. The depth markers, abrasives and finishing instruments are all congruent in size (taper with rounded tip). Two coordinated sizes cover all indications in the entire anterior zone. An egg-shaped diamond abrasive (379) and the matching finishing instrument (8379) allow the preparation of function-correcting palatal veneers.

### Recommendations for use:

- Use preferably in a red contra-angle, observing the speed indicated on the packaging.
- Always supply plenty of spray cooling (at least 50 ml/min.)

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DIAMANT GmbH