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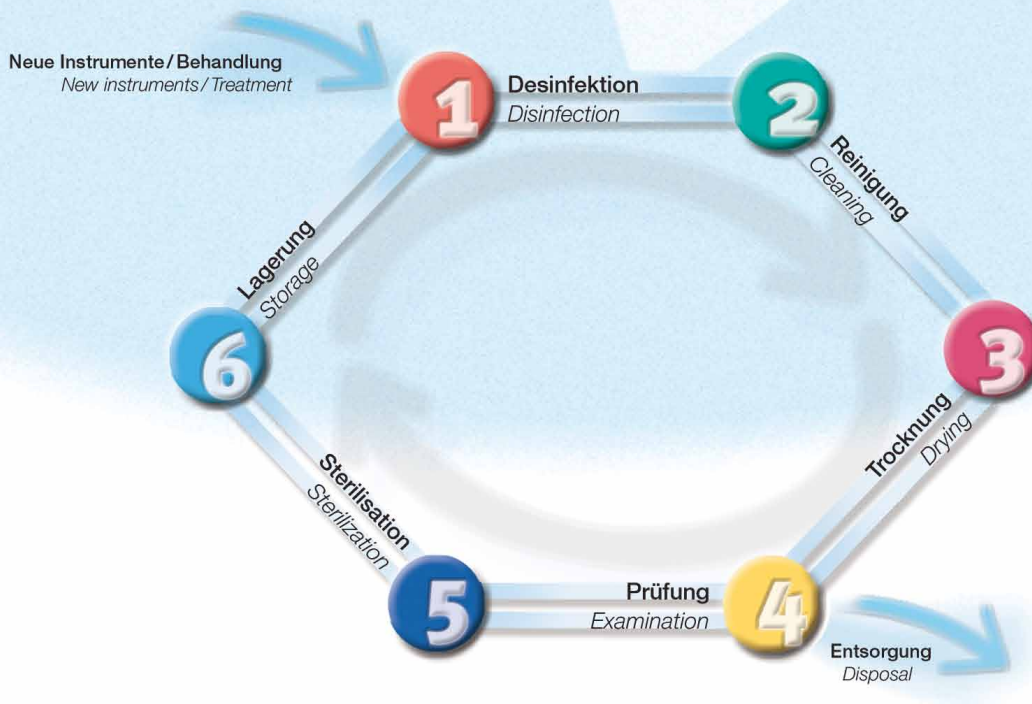
Disinfection, Cleaning, Sterilization

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Disinfection, cleaning and sterilization – Maintenance of instruments in the dental office

The purpose of hygiene measures in the dental office is to preserve health and prevent infectious disease. This graph shows proper disinfection, cleaning and sterilization of dental instruments so that they can be reused.

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→ als Einmalprodukt verwenden

Disinfection

- Use approved agents only (preferably KOMET DC1)
- Immerse instruments immediately after use
- Observe manufacturer's recommendations regarding concentration / time / material-compatibility
- Do not disinfect in the thermodesinfector

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Storage

- Separate storage (sterile/non-sterile)
- Sterile products, 6 weeks max.
- Dry
- Dust-proof
- Separated from chemicals

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Exceptions:

Standard steel instruments (e.g. [REF 1])

→ Instruments corrode if subjected to autoclave sterilization

Rubber polishers

→ change their elasticity if subjected to hot air sterilization

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Reprocessing

A validated cleaning method is available on our web site www.drendel.com under "Instrument reprocessing".

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Cleaning

- Remove residues
 - Cleaning stone **S1000** (for diamond instruments)
 - Brush **B9785** (for blades)
- Ultrasound
- Use anticorrosive cleaning agent
- Thoroughly rinse off cleaning and disinfection agent under running water
- Note:** Use distilled water in order to prevent water spots

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Drying

- Forced drying only with
 - Compressed air
 - Hot air
 - Absorbent paper tissue

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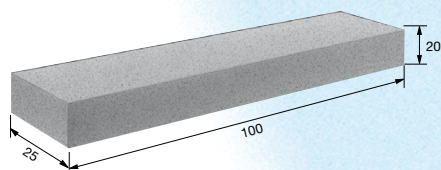
Examination

- Visual inspection
- Disposal of damaged instruments
- Breakouts at the blades
- Dull blades
- Bent instruments
- Dark discoloration
- Corrosion (rust)
- blank spots on diamond instruments

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DIAMANT GmbH