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Tungsten carbide instrument for removing amalgam fillings.

#### 2. Contraindication

There are no known contraindications related to the products.

#### 3. Proper use of the instruments

The instruments are supplied non-sterile and have to be prepared prior to first use.  
- The instrument can be used in the turbine or micromotor.  
- Heat generation and development of mercury vapor is largely reduced when used in the micromotor at a speed of 160.000 rpm.  
- Use low contact pressure.  
- Drill with care to avoid loss of sound tooth substance as the instrument has a highly efficient blade configuration.  
- Make sure that there is always sufficient coolant available. A minimum amount of cooling water of 50 ml/min. is necessary to avoid damage to the pulp and increased mercury vapour.

#### 4. Clinical sequence

- Drilling is carried out step by step  
- Drill into the amalgam filling in axial direction or from an inclined position (Fig. 1).  
- Then cut several longitudinal and transversal separation grooves (Fig. 2). By dividing the filling into segments, fragments of the filling will detach and can be removed from the cavity.  
- Filling residues are removed with suitable hand instruments.

#### 5. Reprocessing

The instruments are supplied non-sterile and have to be prepared prior to first use.  
The instruments are to be disinfected with anti-corrosive disinfection and cleaning agents for rotary instruments (e.g. with Komet DC1). For recommendations for use (immersion time, concentration, suitability) of disinfection and cleaning agents see instructions of the manufacturers of these agents. Disinfecting and cleaning agents must be rinsed off thoroughly with water and the instruments must be dried carefully (preferably with medical compressed air according to the recommendations of the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute). Do

not store instruments for a longer period in wet or humid condition. Make sure that they do not come in contact with each other during ultrasonic cleaning. The instruments can be reprocessed in the thermo disinfectant, provided that the agent used is suitable for rotary instruments (mildly alkaline detergent). Subject cleaned instruments to a visual examination. Damaged or blunt instruments must be rejected and their use discontinued. Sterilization is carried out in the autoclave at 134°C. The recommendations provided by the manufacturer of the respective device must be observed. Discard any corroded instruments. The operator of medical products is responsible for seeing that proper treatment is carried out by qualified personnel, using the appropriate materials and suited equipment, as recommended by the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute. Information on proper reprocessing of instruments according to DIN EN ISO 17664, the present instructions for use, as amended, and more detailed information can be downloaded from our web site [www.drendel.com](http://www.drendel.com) or requested from the manufacturer Drendel+Zweiling DIAMANT GmbH.

#### 6. Storage

Store packed instruments away from UV radiation and heat in a clean and dry environment. Do not store in the same room as solvents or chemicals.

#### 7. Disposal

The instruments have to be disposed of in tight, non-breakable and non-pierceable containers (protection from contamination).

#### 8. Safety and liability

Worn and damaged instruments (bent instruments etc.) have to be discarded and replaced by new ones. The above mentioned recommendations with respect to handling, cooling and contact pressure are to be strictly observed. The instruments should only be used for the intended application. Non-observance of these safety recommendations may lead to damage of the handpiece or injury. The user is responsible to check the product prior to use whether it is suited for the intended purpose. In case of contributory negligence by the user, Drendel+Zweiling DIAMANT GmbH partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

#### 9. Procedure in case of serious adverse events

In case of serious adverse events, please proceed in line with the legal regulations in your country.

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© Drendel+Zweiling DIAMANT GmbH