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EN

Diamond instruments with guide pin

1. Indication

The instruments are used for the following indications:
Crown preparation
Due to the fact that the guide pin is not diamond-coated, a defined preparation line is created.
Therefore, the instruments guarantee controlled and safe preparations.

2. Contraindication

There are no known contraindications related to the products.

3. Proper use

Those instruments that are supplied non-sterile have to be prepared prior to first use.
After the initial preparation of the tooth and dilatation of the sulcus with cord or retraction ring, the instruments with guide pin are moved around the tooth. Shaping is done at an optimum speed of $\varnothing_{rot} 40,000\ \text{rpm}$.
When medium grain or coarse grain instruments are used for shaping, the surface should be finished with a form congruent fine grain instrument at an optimal speed of $\varnothing_{rot} 20,000\ \text{rpm}$.
If necessary, the preparation line should be placed deeper with a form-congruent diamond instrument without guide pin.

Applies to all instruments with guide pin:

- For efficient substance removal, use new instruments.
- Apart from this general information, please observe the individual speed indicated on the packaging.
- Do not exceed the maximum permissible speeds. Otherwise there is a risk of instrument breakage.
- It is recommended to use magnifying glasses during the preparation.
- If possible, apply minimal contact pressure (< 2 N).
- Always use sufficient spray coolant (at least 50 ml/min).

4. Reprocessing

Those instruments that are supplied non-sterile have to be prepared prior to first use.
The instruments are to be disinfected with anti-corrosive cleaning and disinfection agents for rotary instruments (e.g. with Komet DC1). For recommendations for use (immersion time, concentration, suitability) of cleaning and disinfection agents see instructions of the manufacturers of these agents.
Cleaning and disinfection agents must be rinsed off thoroughly with water and the instruments must be dried carefully (preferably with medical compressed air according to the recommendations of the

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Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute). Do not store instruments for a longer period in wet or humid condition. Make sure that they do not come in contact with each other during ultrasonic cleaning. The instruments can be reprocessed in the thermo disinfectant, provided that the agent used is suitable for rotary instruments (mildly alkaline detergent). Subject cleaned instruments to a visual examination. Damaged or blunt instruments must be rejected and their use discontinued. Sterilization is carried out in the autoclave at 134°C. The recommendations provided by the manufacturer of the respective device must be observed. Discard any corroded instruments. The operator of medical products is responsible for seeing that proper treatment is carried out by qualified personnel, using the appropriate materials and suited equipment, as recommended by the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute. Information on proper reprocessing of instruments according to DIN EN ISO 17664, the present instructions for use and more detailed information, as amended, can be downloaded from our web site www.drendel.com or requested from the manufacturer Drendel+Zweiling DIAMANT GmbH.

5. Storage

Store packed instruments away from UV radiation and heat in a clean and dry environment. Do not store in the same room as solvents or chemicals.

6. Disposal

The instruments have to be disposed of in tight, non-breakable and non-pierceable containers (protection from contamination).

7. Safety and liability

Worn and damaged instruments (defective diamond coating, bent instruments, defective guide pin) are to be rejected and replaced by new ones. The above mentioned recommendations with respect to handling, cooling and contact pressure are to be strictly observed. The instruments should only be used for the intended application. Non-observance of these safety recommendations may lead to damage of the handpiece or injury. In case of contributory negligence by the user, Drendel+Zweiling DIAMANT GmbH partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

8. Procedure in case of serious adverse events

In case of serious adverse events, please proceed in line with the legal regulations in your country.

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© Drendel + Zweiling DIAMANT GmbH