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Instructions for use

Please read carefully and retain for future reference.

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EN
Tungsten carbide instrument CB255A, TiNCB255A

1. Indication
- Minimally invasive combination instruments for conservative preparation of bone tissue and hard tooth substance
 - Removal of impacted wisdom teeth
 - Apical ectomy
- The instrument is used for tissue-friendly preparation of bone or bone lids which can be repositioned after removal of impacted wisdom teeth or after apical ectomy and for definite removal of surrounding bone tissue. Due to its efficient cutting ability the instrument is also suited for cutting hard tooth substance, in particular for separating tooth roots.

2. Contraindication
- Treatments where the instrument is used as a lever (e.g. for loosening and removing of prepared bone or wisdom teeth)
 - Not suited for cutting root posts
 - Insufficient external cooling (less than 50 ml/min.)

3. Proper use
- For removal of impacted wisdom teeth or apical ectomy, for treatment of bone surfaces or for cutting bone
- Expose the jaw bone according to the known procedure
 - Prepare the contour of the bone transplant with the combination instrument considering its shape and position or reduce the surrounding bone, applying sufficient external cooling (at least 50 ml/min.). For axial preparation, move the instrument circularly in order to avoid jamming. On one hand the external cooling prevents the bone tissue from being excessively heated and on the other hand it facilitates evacuation and rinsing of bone chips.
 - The preparation is carried out with low contact pressure ($\leq 2N$) to the required depth. Care should be taken that the cutting surfaces are sufficiently exposed to coolant.
 - Separate tooth or root with the combination instrument; with sufficient external coolant, at least 50 ml/min.) and remove the fragments, for instance with forceps or tweezers.

- Treat the wound according to the generally practised procedure and if indicated reposition the bone transplant and close the mucosa.
Note: Instrument shall not be used as a lever risk of accident. Instruments with shank 314/316 are designed to be used in the micromotor (red contra-angle) and not in the turbine.

4. Recommendation for maintenance
- Those instruments that are supplied non-sterile have to be prepared prior to first use. The instruments have to be disinfected with antir corrosive disinfection and cleaning agents for rotary instruments after each use. Recommendations for use of the disinfection and cleaning agents (immersion time, concentration, suitability) are stated in the manufacturer's instructions for these agents. Instruments are to be sterilized in the usual manner. The operator of medical products is responsible for seeing that proper treatment is carried out by qualified personnel, using the appropriate materials and suited equipment, as recommended by the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute. Instructions on the reprocessing of instruments in compliance with the standard DIN EN ISO 17664 can be downloaded from our website www.drendel.de or ordered from the manufacturer Drendel+Zweiling DIAMANT GmbH.

5. Safety and liability
- Worn and damaged instruments must be rejected immediately and replaced by new ones. The instructions for handling, cooling and contact pressure have to be strictly observed. The instruments should only be used for the indicated purpose. Non-observance of these safety instructions may lead to damage of the handpiece or injuries. See also the general recommendations for use and safety instructions in the current D+Z dental catalogue or on our website www.drendel.com. The user is responsible to check the products prior to use as to whether they are suited for the intended purpose. In case of contributory negligence by the user, Drendel + Zweiling DIAMANT GmbH partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

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© 0216.312150 – Rev 02/2016