**General instructions for use and safety recommendations for the application of dental instruments and bur blocks**

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**Area of application**

These general instructions for use and safety recommendations apply to all products and have to be generally observed. Non-observance of these instructions for use and safety recommendations increases the risk of injury and may impair the proper function of the instruments.

Separate instructions of use are available on our web site [www.drendel.com](http://www.drendel.com) for products that require more detailed information. These take precedence over the general instructions.

**Storage**

Instruments in their original packaging, away from light and heat in a clean and dry environment. Do not store in the same room as solvents or chemicals.

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**1. Proper use**

- The instruments have to be prepared prior to first use.
- Make sure that only technically and hygienically perfect and cleaned power systems (hand pieces and contra-angles) are used.
- Depending on their shank type, insert the instrument into the chuck as deeply as possible. Make sure that they are properly locked.
- The instrument must be rotating at the desired speed before contact is made with the tissue or material.
- Avoid jamming and using the instrument as a lever. Excessive contact pressure has to be avoided.
- For eye protection wear safety glasses. Use appropriate respiratory protection (mouth and nose). In the dental laboratory, use appropriate suction unit.
- Avoid contact with the instruments' working parts as this may increase the risk of injury.

Make sure to provide sufficient cooling by means of air/water spray. Additional external cooling is required when using instruments with extra-long shanks or oversized working parts. Damaged or corroded instruments have to be discarded.

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**2. Recommended speeds**

Please make sure to observe the recommendations for use and recommended speeds as indicated in the instructions of use and on the packaging of the products.

- $\varnothing_{\text{max}} 300\,000$  rpm means: Suited for micro-motor contra-angles and turbines with stable ball bearings. Do not use in turbines with air bearing.
- $\varnothing_{\text{max}} 200\,000$  rpm means: Suited for micro-motor hand pieces and contra-angles or lab hand pieces up to the speed indicated. Do not use in turbines.

Not observing the maximum permissible speed leads to an increased safety risk.

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**3. Contact pressure**

Excessive contact pressure (> 2N) has to be avoided.

- In cutting instruments, this can lead to damage to the working part and to chipping of the blades as well as an excessive generation of heat.
- In abrasive instruments, increased contact pressure may lead to stripping of the grit or to clogging of the instruments and increased heat generation.

Increased contact pressure may also lead to thermal damage to the pulp or, in case of damaged blades, to rough surfaces. In the extreme cases, instrument breakage may even occur.

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**4. Guideline on the number of times rotary instruments can be used**

The below values are guidelines. The service life of instruments may differ from these values as this depends on the application and/or the material treated.

Stainless steel instruments:	- 4 x
Tungsten carbide instruments:	- 15 x
Diamond instruments:	- 25 x
Polishers and ceramic abrasives:	- 10 x
Endodontic instruments:	
Wide canals:	- max. 8 x
Average canals:	- max. 4 x
Narrow canals:	just use 1 x

The reuse of disposable products is not permitted.

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**5. Disposal**

To prevent contamination, discard instruments in tight, puncture resistant containers.

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**6. Disinfection, cleaning and sterilisation**

Those instruments that are supplied non-sterile have to be prepared prior to first use. For further information, please refer to our Homepage ▶ Downloads ▶ Manufacturer's information.

**6.1. Manual reprocessing**

The instruments are to be disinfected with appropriate cleaning and disinfecting agents recommended for these products (e. g. with Komet DC1). For recommendations for use (immersion time, concentration, rinsing, drying) of cleaning and disinfecting agents see instructions of the manufacturers of these agents. Make sure that the instruments do not come in contact with each other during ultrasonic cleaning.

**6.2. Mechanical reprocessing**

The instruments have to be treated with suitable detergents and disinfectants (e. g. Komet DC1) that are recommended for this purpose. Observe the instructions of use provided by the manufacturer. Make sure that the instruments do not come in contact with each other during the cleaning in the ultrasonic bath. Inspect the clean instruments visually. Separate and discard damaged or blunt instruments. Thoroughly cleaned instruments are an essential condition for a successful sterilization.

**6.3. Sterilization**

Make sure that the instruments are sterilized according to a validated procedure suitable for the sterilization of medical products. Successful sterilization also depends on the type of product to be sterilized, the packaging and the loading set-up of the sterilization device. The operator of medical products is responsible for seeing that proper treatment is carried out by qualified personnel, using the appropriate materials and suited equipment, as recommended by the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute.

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**7. Specific instructions for individual instrument types**

- Avoid any contact with H<sub>2</sub>O<sub>2</sub> (hydrogen peroxide). Make sure that the specified immersion times in the cleaning and disinfecting agents are not exceeded. The carbide working parts would be attacked (a typical indication is black staining of the instrument) reducing the instrument's service life.
- Tool steel instruments corrode and can therefore not be sterilized with a sterilization method using saturated steam (e. g. autoclave).
- To achieve an optimal surface roughness, subsequent finishing is necessary after using a diamond grinding instrument with coarse or very coarse grit.
- Use a disc guard for diamond discs when working intraorally.
- When using diamond separating strips and diamond strips please avoid contact with the gingiva as there is a risk of injury.
- To avoid overstressing of the instrument, root canal instruments made of nickel-titanium have to be used in a torque limited motor.
- Stainless steel root canal reamers (type "Gates", "Müller") are only intended for preparation of the coronal portion of the canal.

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### 8. Potential effects of using worn instruments

- Discard any instruments with damaged or worn working parts as the use of damaged or worn instruments would have a negative effect on the work result.
- Damaged and deformed cutting blades will cause the instrument to vibrate and lead to poor preparation margins and rough surfaces.
- Void spots on the surface of diamond instruments are a sign of missing diamond particles and reduce the instrument's abrasive efficiency. An inferior diamond coating quality will result in excessive heat generation. Excessive contact pressure or temperatures may cause damage to the tissue.
- Please avoid excessive contact pressure during use as this may result in damage to the working part (Nicks on the blades, premature blunting and excessive heat generation).
- In abrasive instruments, excessive contact pressure can lead to stripping of the grit or clogging of the instruments and increased heat generation.
- To avoid undesirable heat generation during preparation, make sure to provide sufficient cooling by means of air/water spray (at least 50 ml/min).
- Additional external cooling is required when using instruments with a total length of more than 22 mm or a head diameter exceeding 2,5 mm.
- Not observing the maximum permissible speed will result in an increased risk of injury.
- There is an increased risk of infection in reusable instruments which have not been properly reprocessed.
- The reuse of disposable instruments (marked on the packaging) is not permitted (e. g. polishers with lamellae and dental brushes). The reuse of these products poses a risk of infection and/or the safety of the products can no longer be guaranteed (e. g. due to the risk of fracture with root canal instruments).

### 9. Safety and possible side effects

The above mentioned recommendations with respect to cooling, contact pressure, disinfection, cleaning and sterilization are to be strictly observed. The instruments should only be used for the intended application, as per the symbolic identification. Non-observance of these safety recommendations may lead to damage of the power system and/or injury, such as thermal necrosis, undesired preparation of tissue, damage to tissue or nerves, violation of the biological width, or infections. During preparation, some instruments may generate metallic abrasion which may lead to the presence of artifacts in MRI diagnostics.

### 10. Liability

It is the responsibility of the user to check the products prior to use to ensure that they are suited for the intended purpose. In case of contributory negligence by the user, Drendel + Zweilling partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.  
Store products out of children's reach.  
For dental use only.

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